


CEREC CONNECT

HOW TO ADD MDL AS A FAVORITE LAB

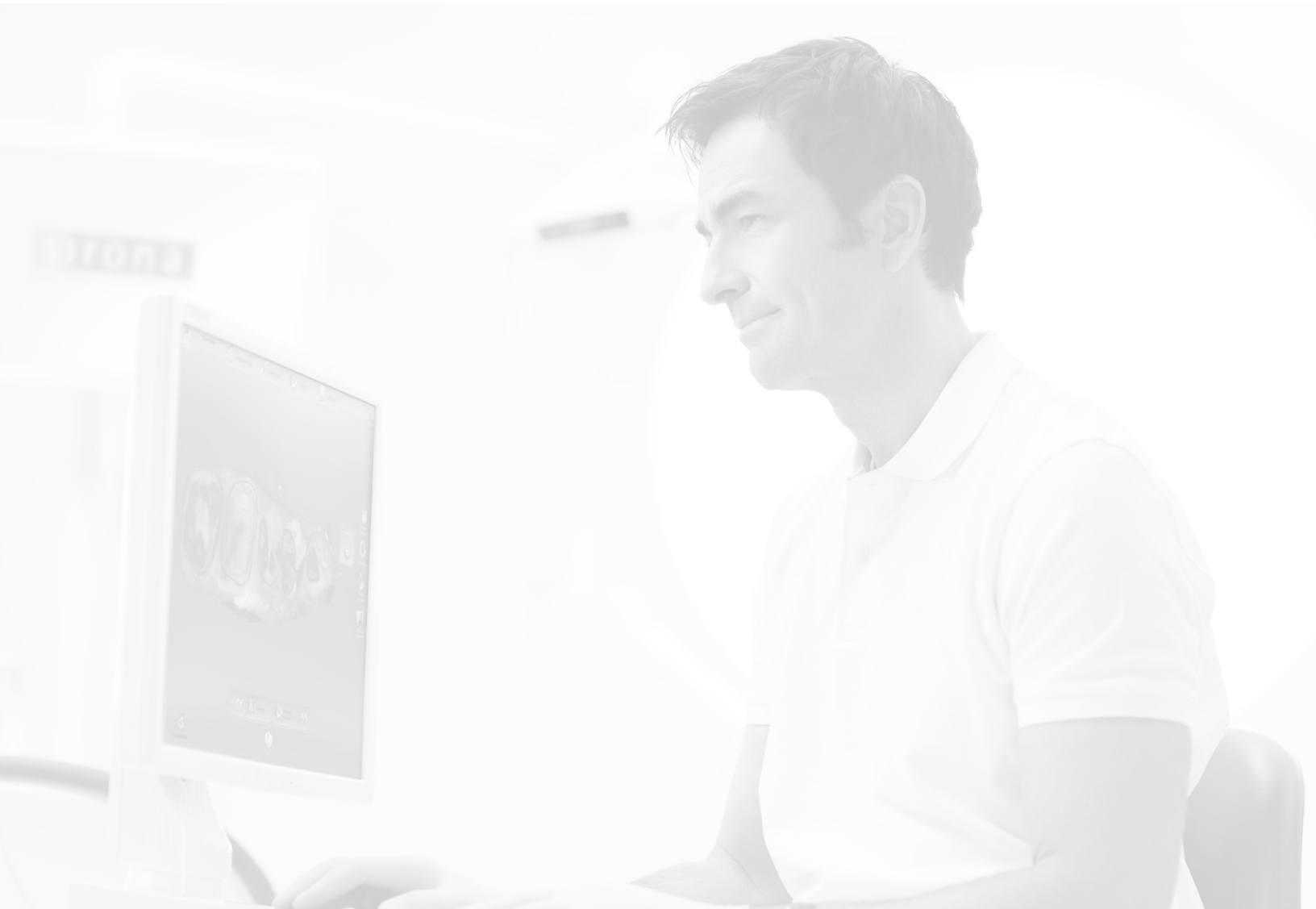


REGISTER YOUR SIRONA INTRAORAL SCANNER

Adding Mountain Dental Lab as a favorite lab in CEREC Connect is quick and easy to do after you have registered your unit online.


- Go to www.sirona-connect.com
- Click on “Dentist Registration”

If you've already registered your unit, please skip ahead to page 6

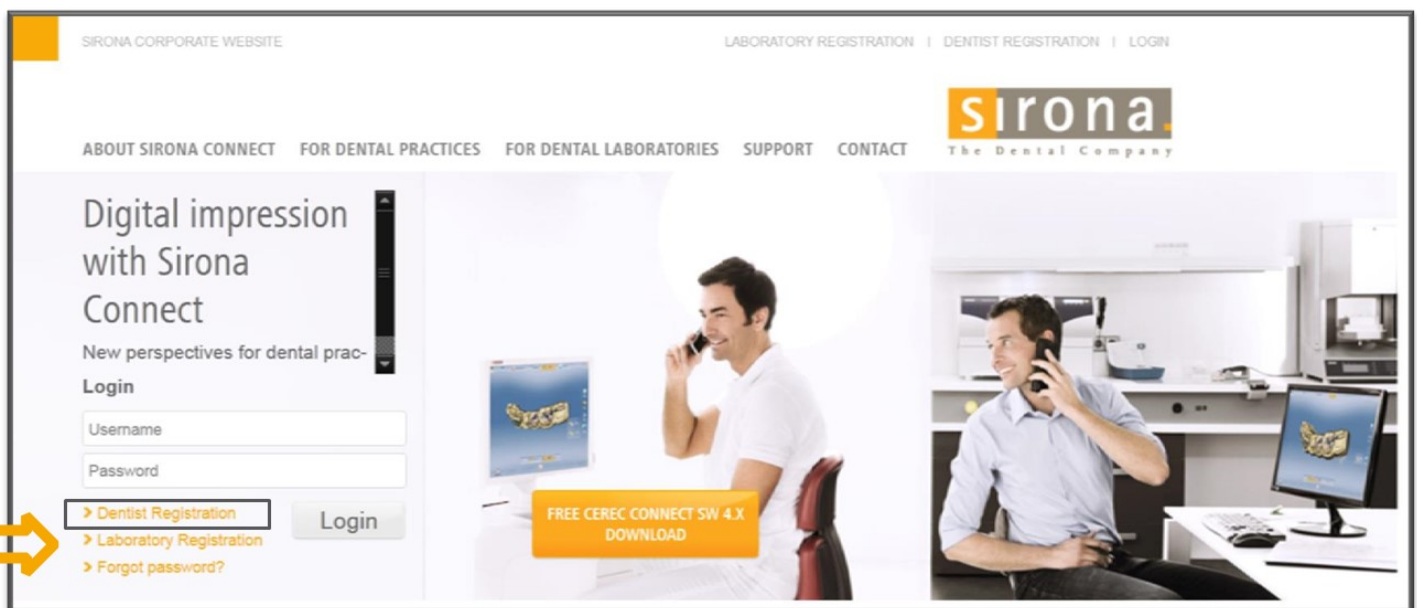

REGISTER YOUR SIRONA INTRAORAL SCANNER

Create a Username and Password, then click the “Next” button

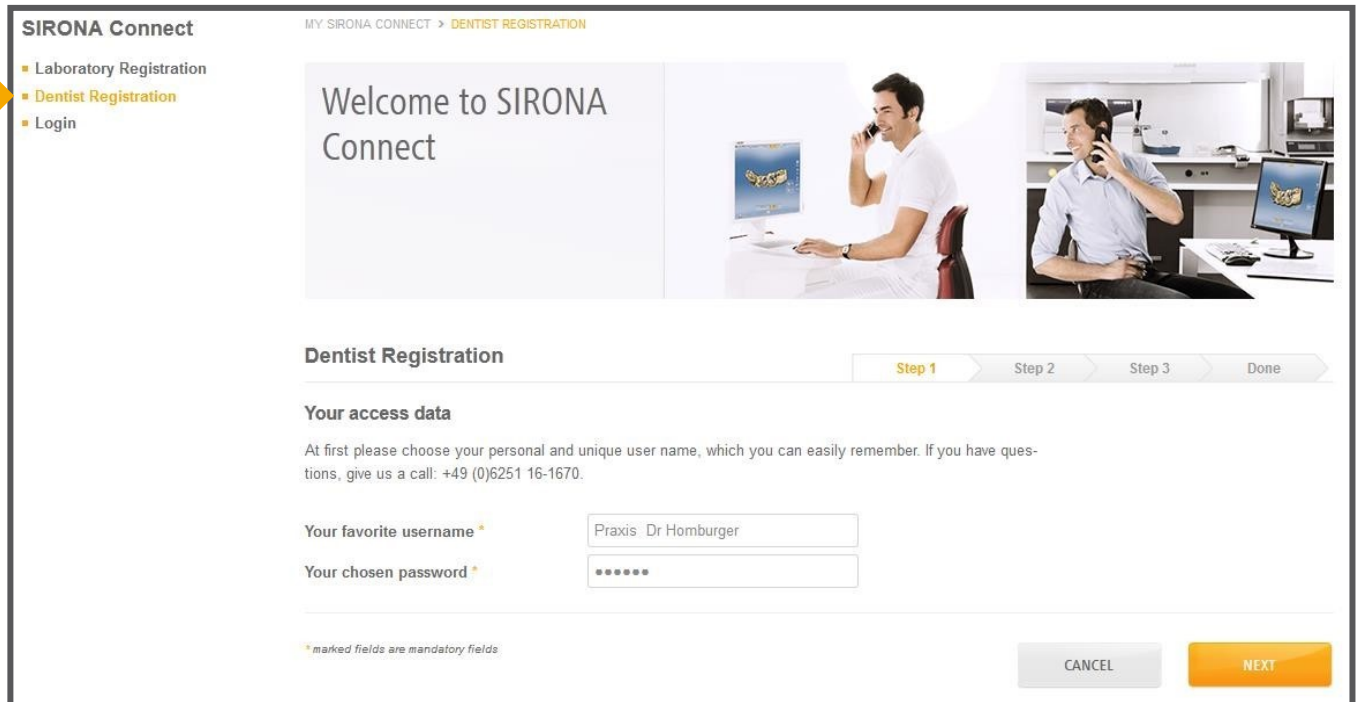

SIRONA Connect MY SIRONA CONNECT > DENTIST REGISTRATION

- Laboratory Registration
- Dentist Registration**
- Login

Welcome to SIRONA Connect

Dentist Registration Step 1 Step 2 Step 3 Done

Your access data

At first please choose your personal and unique user name, which you can easily remember. If you have questions, give us a call: +49 (0)6251 16-1670.


Your favorite username *

Your chosen password *

* marked fields are mandatory fields

CANCEL NEXT

Enter your complete information, then click the “Next” button

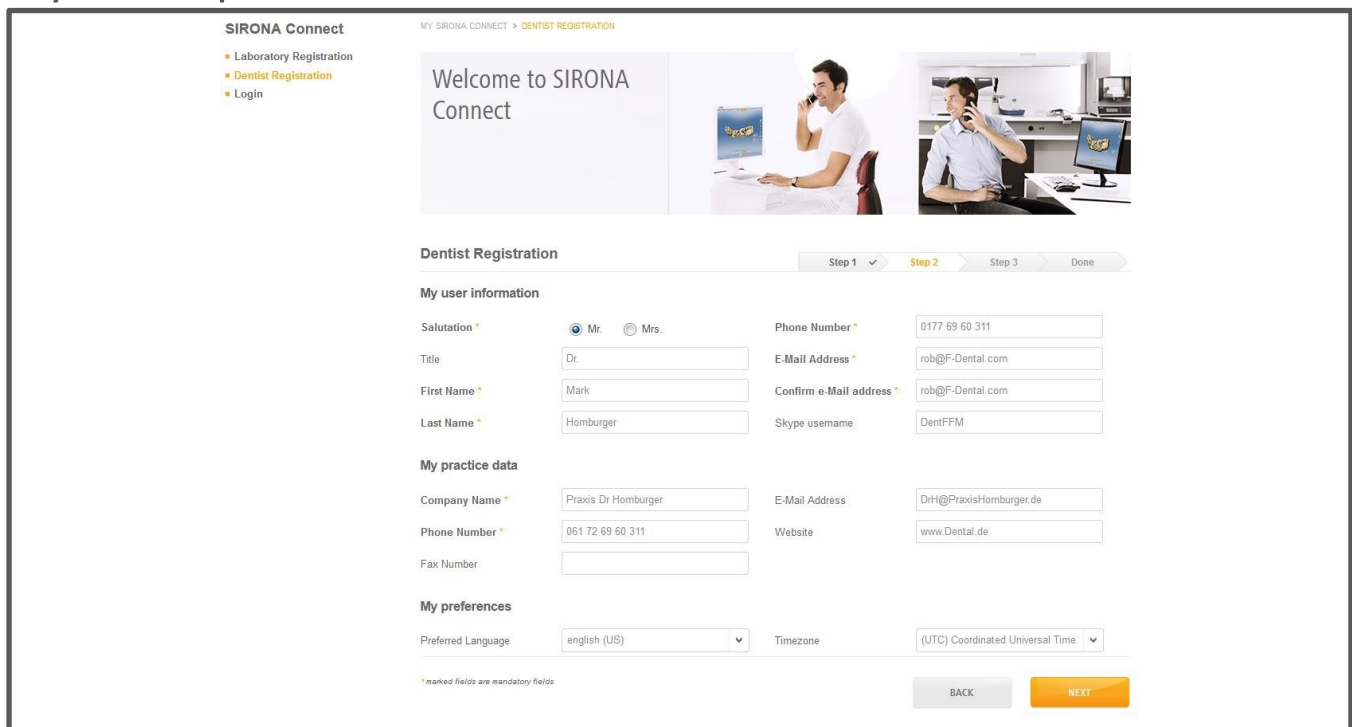

SIRONA Connect MY SIRONA CONNECT > DENTIST REGISTRATION

- Laboratory Registration
- Dentist Registration**
- Login

Welcome to SIRONA Connect

Dentist Registration Step 1 Step 2 Step 3 Done

My user information

Salutation * Mr. Mrs. Phone Number *

Title

First Name *

Last Name *

My practice data

Company Name *

Phone Number *

Fax Number

My preferences

Preferred Language

* marked fields are mandatory fields


BACK NEXT



REGISTER YOUR SIRONA INTRAORAL SCANNER

Enter your Billing and Delivery Address

Welcome to SIRONA Connect



Dentist Registration Step 1 > Step 2 > **Step 3** > Done

My billing address

Address 1 *

Address 2

ZIP Code *

City *

Country *

Region

different shipping address

My shipping address

Recipient

Address 1 *



Address 2

ZIP Code *

City *

Country *

Region





*marked fields are mandatory fields

BACK DONE

A confirmation email will be sent

MY SIRONA CONNECT > **DENTIST REGISTRATION**



Dentist Registration Step 1 > Step 2 > Step 3 > **Done**

Congratulations! You nearly did it.

Your new Sirona account was created. Prior to your first login, we would like to check whether your e-mail address is valid und working.

We have therefore sent you an e-mail with the subject **"Please confirm your e-mail address"** which you can use to activate your account. Therefore please follow the instructions inside the e-mail. You will find a link in the e-mail

If you didn't receive the e-mail, please check your spam folder. If you have problems, call our support (phone number + 49 (0) 6251 / 16-1670) or send us an e-mail (support@sirona.com).

We wish you good luck with Sirona Connect!



REGISTER YOUR SIRONA INTRAORAL SCANNER

Open the confirmation email and click on “Activation Link”

SIRONA Connect – Please activate your account

Von: cerec.connect@sirona.de

An: alex@drauz.net

Datum: 25.03.2014 19:16

Dear Mr. Homburger

Welcome at SIRONA Connect! You have been registered on 03/25/2014 6:16:57 PM.

Your details are:

Username: Praxis Dr Homburger

Dr. Mark Homburger
Phone: 061 72 69 60 311
Email: DrH@PraxisHomburger.de

Dentist data:
Praxis Dr Homburger
Gutleutstr 33

60311 Frankfurt
Germany
Fax:
Website: www.Dental.de

You can activate your account with the following [Activation Link](#)


Sincerely,
Your SIRONA Connect Team

The link will take you to the main Sirona Connect login page. Enter your username and password to verify the activation was successful. If it was not successful, and you are not able to log in, please call Sirona at 1-800-475-5036.



ADD MDL AS YOUR FAVORITE LAB

Log in to your CEREC Connect Portal online

- www.Cerec-Connect.com
- Click on “My Account” and then “Search Labs”
- Enter our Mountain Dental Lab’s zip code: 28803
- To the far right of Mountain Dental Lab, click on the add  button
- You’re now ready to complete an intraoral scan and select Mountain Dental Lab

We are here for you! If at any point you have questions about registering, adding us as a lab, scanning, or sending a case, please give us a call at

855-704-0894 | 828-665-2257

Our goal is to make your investment in digital technology an asset, not a burden. We are excited to partner with you, and thank you for your time.

